Invacare® P550/2 Actio 2™

en	Rollator User Manual	1
de	Rollator Gebrauchsanweisung	5
es	Rollator Manual del usuario	10
fr	Rollator Manuel d'utilisation	14
it	Rollator Manuale d'uso	18
nl	Rollator Gebruiksaanwijzing	22
pt	Andarilho Manual de utilização	26

General

About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING
Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION
Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT
Indicates a hazardous situation that could result in damage to property if it is not avoided.

en

Tips and Recommendations
Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Intended use

The rollator is intended to support a person during walking and may also serve as a resting seat.

The maximum user weight is stated in the chapter "Technical Data" and on the product label.

Indications

The product is intended for support and improvement of a limited walking ability.

The user's remaining walking and cognitive ability needs to be sufficient for safe usage.

Contraindications

The product is not intended for use by persons with severe equilibrium disorders or functional perceptual disorders.

Permissible operating conditions and locations

For indoor use only (in private or public areas).

Not to be used under water in e.g. pools or open-air bath, neither in hot areas as e.g. saunas.

Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

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Invacare Rea AB