Invacare® Banjo™ P452E/3

en	Rollator User Manual	1
de	Rollator Gebrauchsanweisung	4
es	Rollator Manual del usuario	8
fr	Déambulateur Manuel d'utilisation	11
it	Rollator Manuale d'uso	15
nl	Rollator Gebruiksaanwijzing	18
pt	Andarilho Manual de utilização	22

I General

en

I.1 About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

I.2 Intended use

Indications

The rollator is intended for support and improvement of a restricted walking ability. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others.

The rollator is intended to support a person during walking. Furthermore the rollator is intended to carry a load either in a specially designed basket, or on a tray.

The max. user weight is 130 kg. This weight limitation includes the weight of the user and the content of the basket.

I.3 Service Life

The expected service life of this product is five years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

I.4 Warranty

We supply a two year warranty from the date of delivery. Deviations may occur due to local agreements. Damage caused by normal wear to e.g. wheels, handles and brakes or damage caused by abnormal use (e.g. exceeded max. user weight) is not covered by the warranty. Maintenance and operating instructions must be followed for the warranty to be valid.

For manufacturing date, refer to the product label.

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

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Invacare Rea AB